VALTEO
surgical excellence

GO ORTHO

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Description: The orthopedic gilet is used to immobilise an upper limb in the «elbow against body» position, with an adjustable elbow flexure angle. It is reversible and can therefore be used either on the left or the right side.

Indications: Shoulder traumatology: dislocation, fractures (head of humerus, greater tuberosity of humerus, coil), shoulder separation, fractured clavicle.

Rheumatology: Periarthritis, acute frozen shoulder.

Neurology: Paralysis of the brachial plexus.

Post-operative: Rotator cuff surgery, acromioclavicular articulation or clavicular surgery. Re-education.

Use: The gilet should be worn over a cotton undergarment. The gilet should be fitted by a qualified person and must not be placed in direct contact with a wound. The gilet is packaged for left shoulder fitting. To fit it on the right shoulder, simply turn it over.

VALTEO SA
Case postale
Route de Lausanne 47
CH - 1110 Morges
T +41 22 588 96 10
F +41 22 588 96 20
info@valteo.ch

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Adjust the padded shoulder strap 1 on the shoulder to be immobilised.

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Connect back section with section 2 and fasten it on the chest.

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Position the forearm at the required angle and close flap 3.

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Adjust the elbow support and close the flap under the arm.

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Immobilise the arm by using the flap.

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